


### Layer thicknesses and preparation guidelines

To ensure clinical success of restorations made from VITA ENAMIC, the following **minimum layer thicknesses** must be adhered to:

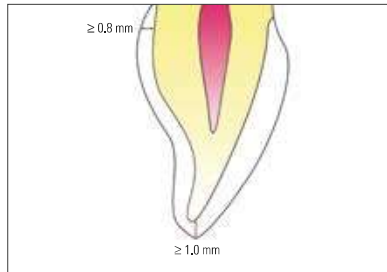


#### Anterior crowns

Incisal: **at least 1.0 mm**  
Circumferential: **at least 0.8 mm**




#### Posterior crowns

Occlusal: **at least 1.0 mm**  
Circumferential: **at least 0.8 mm**

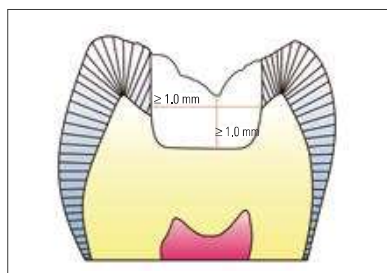


#### Inlays

Occlusal: **at least 1.0 mm**  
In the area of the isthmus: **at least 1.0 mm**

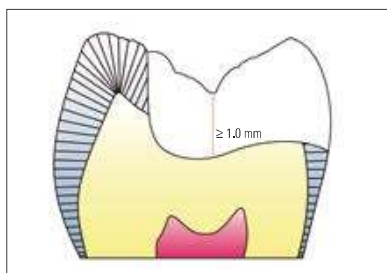


#### Onlays

Occlusal: **at least 1.0 mm**

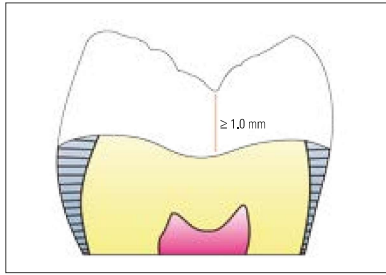

**Table tops**

Occlusal: **at least 1.0 mm**

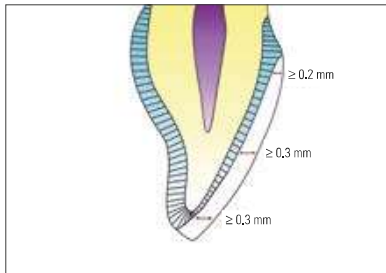

**Veneers**

Incisal: **at least 0.3 mm**

Labial: **at least 0.3 mm**

Cervical: **at least 0.2 mm**